

ACCURATELY DESIGNED FOR PATIENT EASE!



ViSure TRANSPORT KIT

- ViSure transport kit is intended for the collection and transport of clinical specimens
- Recommended by CDC & WHO for collection and transport of Coronavirus
- Storage : 15-30° C before sample collection 2-8° C after sample collection
- Based on recommendations of CDC and specifications of ICMR.
- Users include ICMR authorized labs


ViSure Transport Kit

ViSure transport kit is intended for the collection and transport of clinical specimens containing viruses from the collection site to the processing laboratory.

Sample Collection


Samples for research on viruses should be collected and handled with the following steps:

A. For Nasal specimen

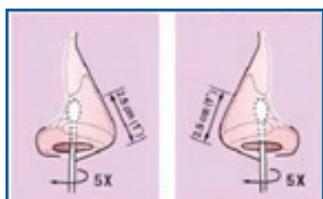


A dry polyester swab is inserted into the nostril, parallel to the palate, and left in place for a few seconds. It is then slowly withdrawn with a rotating motion. Specimens from both nostrils are obtained with the same swab

B. For Nasopharyngeal specimen

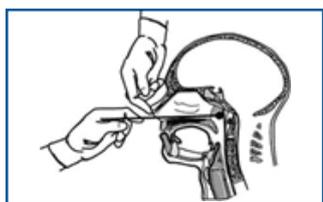

A flexible, fine-shafted polyester swab is inserted into the nostril and back to the nasopharynx and left in place for a few seconds. It is then slowly withdrawn with a rotating motion. A second swab should be used for the second nostril.

C. For Throat specimen

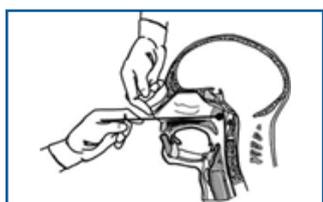

- A. Ask patient to open his/her mouth. Swab the back of throat near the tonsils
- B. After specimen collection, break the swab near the break point and insert into the transportation tube containing viral transport medium and close the cap tightly.

Kit Information

Sr. No	Components	Composition
1	Viral Transport Medium*	Proprietary
2	Double Sterile Nylon Flocked Swabs	Proprietary

Ordering Information

Product Code	Product Description	Pack Size	Storage	Shelf Life
30201273	ViSure Transport Kit	50 Tubes & 2 Swabs/test	Room Temperature 15-30°C	12 Months

*Please note: Product specifications are subject to change without prior notice owing to product modifications, improvements / up-gradation.

corporate@trivitron.com | www.trivitron.com

Customer Care Helpline: +91 98400 80008

ViSure Transport Kit



TRIVITRON
HEALTHCARE
speaking your language