



Meril

Diagnostics



# MERIL VIRAL TRANSPORT MEDIUM KIT

Meril Viral Transport Medium is used for collection, transportation and preservation of human nasopharyngeal or oropharyngeal virus samples from the collection site to the testing laboratory.

## Features:

- Best suited for the transportation of viruses in an active form
- Maintains the virulence and viability of the viral sample
- Contains protective protein, antibiotics and buffers
- Provide appropriate transport and storage environment for swab samples
- Contains a cryoprotectant for preservation (frozen) for prolonged storage



## Procedure:

### Nasopharyngeal swab:



Gently insert the swab into patient's nasal condyle of the nasal passage and rotate the swab for 2-3 times, hold for few seconds and remove slowly

### Oropharyngeal swab:



Gently insert the swab into patient's throat and wipe the bilateral pharyngeal tonsils and posterior pharyngeal, hold for few seconds and remove slowly.

Open the tube and insert the swab into the tube

Immerse the swab head into the medium

Break the swab at break point

Discard the tail end of the swab

Recap the tube tightly

Label the tube for identification and storage

Transport at 2° - 8°C.



## Ordering Information:

Product Code	Material Description	Pack Size
MBVTM-01	Meril Viral Transport Medium Kit	50 Test



### Meril Diagnostics Pvt. Ltd.

D1 - D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

601, Midas, Sahar Plaza Complex, J.B Nagar, Andheri East, Mumbai - 400059

Tel: +91 22 30732400 | Fax: +91 22 39350777 | Customer Care: 1-800-266-3745 (Toll Free) | Email: [diagnostics@merillife.com](mailto:diagnostics@merillife.com)