


\*Read whole instructions carefully before use!

## IFU for KRIVET Insulin Pen Needles

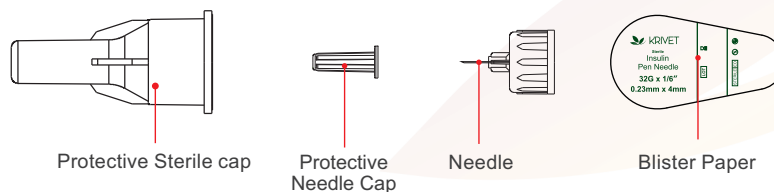

1. Remove protective blister paper.
2. Press the insulin pen straightly down on to the needle and screw until locked.
3. Remove and save the protective sterile cap for future use.
4. Remove the protective needle cap.
5. Inject insulin according to your physician's instruction.
6. Recap the protective sterile cap & unscrew from insulin pen and dispose-off the used needle with the cap.

### Warning

1. If the individual packaging is damaged, or if the seal is missing, or if the needle appears to be bent or damaged, kindly do not use the needle to prevent infection.
2. Remove both the protective sterile cap and the protective needle cap before an injection. If both the outer cover and the inner needle cover are not removed before use, the medication or dose may not be injected, which may be dangerous.
3. Make sure to inject the prescribed dose of insulin. Check the flow of insulin according to the instructions for your insulin pen.
4. Once the needle has penetrated into the skin surface, do not change the angle of the insulin pen. It may cause the needle to bend or break off.
5. Keep the insulin pen needles out of the reach of children.
6. Use the needle only once. Always use a new needle for each injection. Today's insulin needle is thinner and more delicate for greater comfort, so needle re-use can damage the tip and cause injury. Use only once. Needle re-use can cause the tip to bend, causing bleeding, bruising, or scarring and can cause infection.











**\*Read whole instructions carefully before use!**

## Caution

1. In according to the instructions of your physician's, change your injection site regularly to prevent hardening of the tissue.
2. Store the needle in a dry and cool environment.
3. If insulin continues to drip from the needle after injection, hold the needle in the skin longer about 10 seconds to complete to dose.
4. Always prime your insulin pen before each injection. Dial two units on your pen and then press the button to shoot some insulin into the air to make sure it works. If you do not see at least two drops of insulin after repeated priming, do not use the pen. This indicates there may be a problem with the pen. Sometimes when you put a needle on a pen, a drop of insulin may leak out. This does not mean the pen has been primed. You will still need to prime before you inject.
5. After your injection put your used needles in a FDA-cleared sharps disposal container right away after use.

Do not dispose the needles in your household trash.

## Symbols Used:

Symbol	Meaning	Symbol	Meaning
	Date of Manufacture		Storage temperature
	Lot Number		Do not re-sterilize
	Manufacturer		Expiry date
	Catalogue number		Do not open, if seal is broken
	Sterilized using ethylene oxide		Do not re-use