


*Read whole instructions carefully before use!

IFU for KRIVET Insulin Syringes

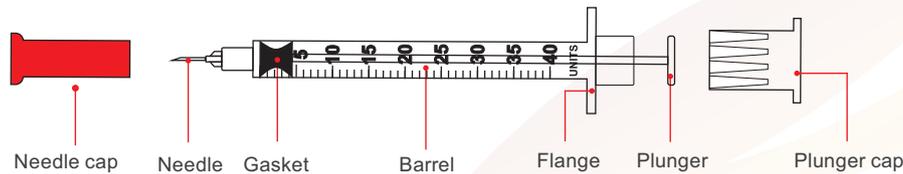

Preparing the Dose:

After washing your hands, take the Insulin vial out of the refrigerator. Shake the vial until the insulin is uniformly mixed.

1. Carefully remove both the protective caps (needle cap and plunger cap) and insert the needle into the vial.
2. Turn the vial upside down, making sure the tip of the needle remains in the vial.
3. Withdraw the correct dose into the insulin syringe.
4. Before removing the needle from the vial, check the syringe for any air bubbles.
5. If bubbles are present, hold the syringe up and tap its side until the bubbles float to the top. Push them out with the plunger and withdraw the correct dose.
6. Remove the needle from the vial.

Giving the Injection:

1. Injection should be given under the skin, 3/4 to 2 inches from the middle of your pet's back, varying from behind the shoulder blades to the middle of the back region and alternating sides. (Consult your Veterinarian for the perfect site and procedure)
2. Gently insert needle in the centre of folded skin.
3. Push plunger until all of the insulin is given.
4. Pull needle out and dispose of syringe into appropriate sharps collector.

*Read whole instructions carefully before use!

Injection Tips

- Injection site can depend upon what is best for the pet.
- Alternate injection site between left and right side for comfort and best results. Repeat injections at the same site can lead to ineffective treatment
- Recording the location of administration can help assure site rotation.
- If only a partial dose is given, do not attempt to re-dose. Wait and give the pet its usual dose at the next injection time.

Symbols Used

Symbol	Meaning	Symbol	Meaning
	Date of Manufacture		Storage temperature
	Lot Number		Do not re-sterilize
	Manufacturer		Expiry date
	Catalogue number		Do not open, if seal is broken
	Sterilized using ethylene oxide		Do not re-use

